DeSensiDent is intended to relieve tooth sensitivity that you get from cold, heat, acids and sweets.

PRODUCT DESCRIPTION

DeSensiDent has two key components, which when used together, will effectively reduce sensitivity for an extended period of time. The first component, DeSensiDent tooth serum, is to be brushed on twice a day, replacing your current toothpaste. The second component, the DeSensiDent nHAp™ sponges, are then applied to the areas of sensitivity once a week.

HOW DOES DESENSIDENT HELP TO RELIEVE TOOTH SENSITIVITY?

Tooth sensitivity is caused by damaged enamel and receding gums, which leads to exposed dentinal tubules. These exposed areas allow fluid to move more freely within the tubules, pulling or pushing on your nerves. The result of this fluid movement, is the nerve sending a signal to the brain, that you perceive as tooth sensitivity. DeSensiDent uses a patented nano-Hydroxyapatite (nHAp™) formula to remineralize the exposed dentin tubules and repair the damaged enamel. The result of the remineralization is repair of the enamel, by covering the open tubules, which relieves sensitivity.

SAFETY WARNINGS

If irritation occurs, discontinue use and seek advice from your physician or contact your local Poison Control Center. Do not use product if you are allergic to any of the ingredients listed on the box.

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

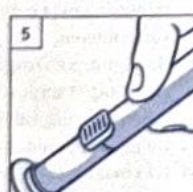
nHAp™ DeSensiDent
SENSITIVITY RELIEF

MANUFACTURED IN THE NETHERLANDS
BY PREVDENT INTERNATIONAL BV

- 1 Replace your current toothpaste and brush your teeth twice a day with the DeSensiDent nHAp™ Tooth Serum and insure you brush just prior to the nHAp™ application.
- 2 Identify the teeth that are giving you sensitivity.
- 3 Once a week, take a fresh DeSensiDent nHAp™ Repairing Serum sponge and apply the nHAp™ formula to your sensitive teeth.
- 4 To release the nHAp™ fluid, place tube between your thumb and index finger. The thumb should rest on the ribbed plastic block.
- 5 Gently press with the thumb on the ribbed plastic block to break the internal lock, which will then release the fluid, to be soaked up by the sponge.
- 6 Squeeze the tube and allow the sponge to soak up the fluid. Once the sponge has soaked up the fluid, apply the fluid to all the sensitive teeth by dabbing the sponge on the tooth surface of the sensitive teeth.
- 7 For optimal results, the formula should be applied and then allowed to sit on the tooth surface for 2 - 3 minutes. Do your best to avoid saliva washing the formula away during this time.
- 8 After 20 minutes you can rinse your mouth with water.
- 9 The nHAp™ Repairing Serum sponges are designed to only be used once, but can be used on all your sensitive teeth during that one application.
- 10 Total treatment time, one month.
- 11 Once treatment is completed, continue your enamel and sensitivity management by using DeSensiDent once every three months and replace your current toothpaste with the nHAp™ Toothpaste for daily use.

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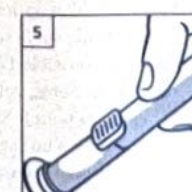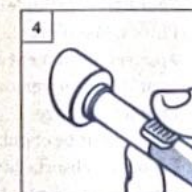Bijl. 28.230.v2

For more Excellent Oral Care products visit
www.prevdentshop.com or contact your local dealer
Prevdent International B.V.
1752 PB IJpendam
The Netherlands
T +31 (0)20 66 177 44
E info@prevdent.com
W www.prevdent.com

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